promed 2520

Promed GmbH

Lindenweg 11

D-82490 Farchant

Tel: +49 (0)8821/9621-0

Fax: +49 (0)8821/9621-21

info@promed.de

www.promed.de

Gebrauchsanleitung | Instruction leaflet | Instruction d'utilisation | Instrucciones de empleo | Istruzioni per l'uso
Gebruiksaanwijzing | Руководство по эксплуатации | Instrukcja użytkowania | Bruksanvisning | Käyttöohje

promed 2520

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SE

FI

- A** Power cord
- B** Socket for the hand unit cable
- C** On/off switch
- D** Speed controller
- E** Tension ring (chuck open/closed)
- F** Switch for direction of rotation, counter-clockwise/clockwise
- G** Socket for the foot pedal cable
- H** Foot pedal
- I** Control LED
- J** Fuse
- K** Voltage selector switch

Important Notes:

Please hold at least 2 - 3 mm distance between handpiece point and drill bits.

Before drill bit changing, please reduce the rotation speed to zero. Otherwise the handpiece can be damaged.

DESCRIPTION OF THE UNIT

Please keep this user manual in a safe place!

Please read the user manual thoroughly before using the appliance for the first time.

Thank you for purchasing the **Promed 2520** manicure and pedicure device. This device was designed and made to guarantee the quality of treatment and may be used by both qualified personnel and private individuals after reading these instructions. This device is designed for use in the manicure and pedicure treatments.

We, as the manufacturer, cannot be made liable in any way for injury or damages to people or objects that arise from failure to comply with this warning. The unit is also suitable for use in the area of nail techniques and nail design as well as the handling of all materials used in this area such as gel and acrylic.

We wish you a lot of enjoyment with your new **Promed 2520** unit. We would like to familiarize you with the unit in the following sections. Please read the user manual thoroughly before using the unit for the first time.

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

INFORMATION ON THE UNIT / ACCESSORIES

Before operating the unit, please check that all contents have been delivered and that the line voltage is correct. The voltage selector switch must be in the 220 V (**K1**) position when the line voltage is 220 V and connected to a properly installed wall socket, and in the 110 V (**K2**) position when the line voltage is 110 V.

- The **Promed 2520** unit, in combination with the grinding heads designed for this unit, is ideal for manicures, pedicures, and mobile pedicure services, as well as for nail techniques (nail design) in particular.
- The **Promed 2520** unit uses a high performance motor with airflow cooling. The electronic speed control is used to smoothly set the speed of the grinding head up to 20,000 rpm. The unit can rotate clockwise and counter-clockwise (reversible).
- The connector for the file attachments is compatible with all commercially available attachments with a shaft of the standard sizes 2,32 mm to 2,35 mm. To ensure that the connector has as long a life as possible, please use only high-

quality attachments with a rounded shaft end and a maximum shaft length of 36 mm! When putting down the hand unit, make sure that the file attachment does not come into contact with its surroundings, otherwise there is a danger that the shaft may be compressed/twisted. An out-of-balance attachment will cause damage to your hand unit!

- File and sanding attachments are available from your local specialist dealer or from **Promed** directly.
- To ensure hygienic operation, the grinding heads must be disinfected after use.
- The handpiece may only be disinfected using disinfectant wipes.

Information about suitable disinfectants and products can be obtained from **Promed**.

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

OPERATING THE UNIT

Before you switch the unit on, you must set the rotating speed (rotary knob **(D)**) to zero.

The power plug is intended as a separator. Make sure that the power plug is plugged into an easily reachable wall socket.

- A** Before operating this unit, please ensure that the voltage specified on the unit corresponds to the local line voltage. Insert the power plug on the power cord **(A)** into the wall socket.
- B** Make sure now that the unit's toggle switch „POWER“ **(C)** is in the „off“ position (the control LED **(I)** should not light up yet).
- C** Set the rotation speed **(D)** of the unit to „0“.
- D** Insert the hand unit's plug into the main unit's connecting socket **(B)**.

- E** Select a suitable grinding head. Open the chuck by turning the tension ring **(E)** anti-clockwise. Insert the grinding head into its holder at the front of the hand unit. Lock the chuck by turning the tension ring **(E)** clockwise. Check the position of the grinding head and whether the chuck on the hand unit is properly locked. Never use force to remove or insert a grinding head! Grinding heads with 2.32 mm to 2.35 mm shaft diameter fit exactly into the grinding head holder without you having to use excessive force. To remove the grinding head, please turn off the unit.
- F** To turn on the unit, pick up the hand unit and push the toggle switch **(C)** onto the „on“ position (the unit's control LED **(I)** must light up green when the unit is turned on).

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

OPERATING THE UNIT

G Adjusting the direction of rotation and speed:

1. Selecting the direction of rotation:

This can be selected by using the toggle switch **(F)**;
R = clockwise or L = counter-clockwise.

2. Selecting the speed:

The speed can be smoothly adjusted to the desired level by turning the speed controller **(D)**.

H IMPORTANT:

If you wish to change the direction of rotation **(F)**, it is imperative that you turn off the unit using the toggle switch „POWER“ **(C)**.

I IMPORTANT:

When changing grinding heads, it is likewise necessary that the speed controller **(D)** either be placed in the „0“ speed position or the unit turned off at the on/off switch.

- J** A foot pedal is included in the **Promed 2520** delivery contents. When using the foot pedal, attach the pedal to its connecting socket „FOOT CONTROL“ **(G)** on the rear side of the unit. The toggle switch „POWER“ **(C)** must be in the „off“ position. You can now set the desired speed of rotation **(D)**. It is now possible for you to turn the unit on and off using the foot switch **(H)**.

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

OPERATING THE UNIT

IN CASES OF MALFUNCTIONING

- Check that the power supply is correct.
- Check whether the hand unit is connected correctly to the main unit.
- Check whether the unit is switched „on“ (the green control LED (I) must be glowing).
- Check whether the fuse (J) is still intact.

The fuse is to be found on the rear side of the unit. Before changing the fuse, please be sure to unplug the unit from the mains outlet. To change the fuse, use a screwdriver to unscrew the fuse holder and replace the micro fuse (T 2 A / 250 V).

- If the problems have still not been solved after following these steps, please consult your service center immediately.

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

MAINTENANCE TIPS

- A.** Keep the inner parts of the handpiece as free as possible from dust and grinding residue.
- B.** Remove dirt from the outer surface of the handpiece with a cloth moistened with alcohol. Prevent moisture from entering the unit.
- C.** Only use grinding heads with a dental standard size of 2.32 - 2.35 mm (0.091 - 0.093 inches) and avoid using rusted or damaged grinding heads. Their false usage may lead to excessive wear and tear in the ball bearings. Adhere to the safe speed and the safety measures stipulated by the manufacturer.
- D.** Do not expose the handpiece to strong knocks as this may damage the ball bearings. If the handpiece should fall down, check whether the grinding head is bent. Pay closer attention to noise and/or heat generation. Send the handpiece immediately in for repair should it be damaged.

Changing the carbon brushes

The carbon brushes in the hand unit motor should be changed once a year by qualified service personnel, either via your dealer or **Promed** directly.

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

SAFETY INSTRUCTIONS

Please pay attention to the following before first time operation:

Never place the unit in direct contact with fire, gas, oxygen or hot objects such as hot plates.

Please observe the following during operation:

- A.** The handpiece may only be connected to the unit included in the delivery contents.
- B.** Grinding heads to be used:
 - Standardized dental size: 2.32 - 2.35 mm / 0.091 - 0.093 inches.
 - Only use centered grinding heads with a rounded shaft end.
 - The use of rusted or bent grinding heads is not permitted.
- C.** Use the speed controller carefully.
- D.** Switch the unit off before you change the direction of rotation.
- E.** Before you switch the unit on, you must set the rotating speed (rotary knob **(E)**) to zero.
- F.** Appliance power supply: 115 V~ or 230 V~.
- G.** Switch the unit off before you change the grinding head.
- H.** Take every precaution possible to ensure that the unit does not fall down or get damaged in any other way.
- I.** If problems do arise with the handpiece, take it immediately to get it repaired.
- J.** Do not lubricate or wash the handpiece.
- K.** Do not switch the unit on while the chuck is opened.
- L.** Place the test grinding head in the chuck when storing the handpiece.
- M.** Never remove the dust covers from the switches as we can otherwise no longer guarantee that your unit will function properly.
- N.** Only for home use.
- O.** Scrupulous supervision is required when the device is being used in the vicinity of children.

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

SAFETY INSTRUCTIONS

DANGER!

- Do not place the unit in direct contact with water or other fluids.
-  Never place or use the unit in a wet or moist environment.
- Do not touch the unit with wet hands.
- Only operate the unit with the specified line voltage.
- It is not permitted to use the unit outdoors.

WARNING!

- Make sure that children do not play with the device.
- Never leave the unit unattended when children or inexperienced persons are in its vicinity.
- Only use the unit for the applications described in this manual.
- Only use attachments and accessories from the manufacturer.
- Never use the unit after the electrical lead has been damaged,

when the unit has fallen down or when it has come in contact with water.

- This unit is not designed to be used by people (including children) who have limited physical, sensory, or intellectual abilities or who do not have sufficient experience and/or sufficient knowledge. This does not apply if these people are supervised by a person responsible for their safety or receive instructions from them on how to use this unit.
- Never let children play with the packaging material; there is a risk of suffocation.
- An electrical appliance is not a toy. Children do not recognize the risks that could develop when using electrical appliances. Therefore, operate and store the unit out of the reach of children.
- Do not place or use the unit on hot surfaces or near naked gas flames to ensure that the casing does not melt.

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

SAFETY INSTRUCTIONS

- Never let the unit come into contact with water. Use a moist cloth to clean the unit. Do not use solvents.
- Never carry or pull the unit by the power cord. Use the plug to pull the power cord out of the wall socket. Do not pull on the power cord itself.
- Do not pull the power cord over sharp edges, wedge it in between objects, hang it down or position or hang it over hot objects (e.g., hot plates) or naked flames. Protect the power cord from heat or oil.
- Only use extension cords when you have satisfied yourself beforehand that they are not defect or damaged.
- If the unit's power cord should become damaged, always replace it with a new one. (Available in electrical specialist stores).
- Do not store the unit near a wash basin or bath tub as there is the danger that the unit may fall or be pulled into the wash basin or the bath tub.
- To avoid all risks of injury, blockage or damage to the unit, make sure that the unit does not come into contact with clothing, cords, shoelaces, bristles, wires, threads, etc. when in operation.

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

SERVICE

Your customers will definitely be expecting you to deliver a professional result. This is only possible with an appliance that is neither defect nor damaged. Therefore, take advantage of our service and have your appliance regularly examined by our specialists. Send in your appliance to Customer Service every 12 months at the latest. We would be happy to provide you with a quote for the costs charged for appliance servicing.

DISPOSAL

Power tools, accessories, and packaging should be recycled in an environmentally suitable manner.

Only for EU countries:

Do not dispose of power tools with the domestic waste! In accordance with the European Directive 2002/96/EEC on waste electrical and electronic equipment and its implementation into national law, nonfunctioning power tools must be collected separately and recycled in an environmentally suitable manner.

This symbol indicates that disposal of this product in the domestic waste is not permitted within the EU. Waste devices contain valuable recyclable material that should be recycled. Recycling also prevents uncontrolled waste disposal from damaging the environment and human health. Therefore, please dispose of all waste devices in the appropriate collection systems or send the device to the place where you bought the device for disposal. They will then recycle the device.

DE

US

FR

IT

ES

NL

RU

PL

SE

FI

WARRANTY

This unit has been manufactured with the utmost of care and has been examined in detail before leaving the factory. We therefore provide a warranty of 24 months after the date of purchase when presented with the warranty card issued with this appliance, in accordance with the following conditions.

- Within the warranty period, we will replace all defect parts of the appliance free of charge, including labor costs for repairs under warranty, in cases of verifiable faults in material or manufacture that have occurred with correct usage and have been identified during the warranty period.
- This warranty is void when the unit has been improperly used (e.g., connection to unsuitable sources of electricity, breakage), the unit has been tampered with (e.g., opening the unit's casing), and when spare parts have been used that have not been approved of by **Promed**. Parts subject to wear are exempt from the warranty. Appliance motors and mobile parts are not included in the warranty.
- The warranty period begins on the date of purchase. Warranty claims do not affect the warranty duration. Warranty claims must be lodged within the warranty period. Claims arising after cessation of the warranty period cannot be considered.
- The warranty only comes into effect within the scope of these warranty conditions when the date of purchase has been confirmed by the stamp/ signature of the dealer on the warranty card.
- In cases of warranty or repair claims, please post the complete appliance with the completed warranty card to the customer service point responsible for you. The warranty card can be found at the end of this user manual.
- We reserve the right to make technical and optical changes as well as changes to the equipment!

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Symbol for protection class I appliances

The symbol corresponds to the symbol 5019 of the IEC 60417 standard. All conductive housing parts in the device are connected to the protective bonding circuit of the fixed electrical installation located at the ground potential.

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Symbol for WARNING! Indicates sections of the user manual that are relevant for safety!

The symbol corresponds to symbol 14, table D1, annex D of the EN 60601-1:1990 Standard. Instructions for an appliance, system or accessory that contain important information for the user that are relevant for safety.

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Pay attention to the symbol for the user manual!

The symbol corresponds to the D-M018 symbol in the DIN 4844-2 Standard. It is imperative that the manual be read before using the unit.

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